

AmpSafe™ Quick Start Guide

Power on the AmpSafe™ unit and wait for the LED to blink green

You can choose a wired or wireless connection...follow one of the chosen paths below.

Wired Wireless



- Connect the USB cable to the AmpSafe™ unit and to the BYOD as shown in the photo above.
- Open a Web browser on the BYOD and navigate to http://sn000000.ampsafe (where "00000" is the serial number of the unit which can be found on the unit's enclosure).
- If this does not connect, enter the following numeric IP address in the Web browser: http://192.168.42.1



- The AmpSafe[™] unit will broadcast a network name (SSID)
 of "AmpSafe-SN00000" (where "00000" is the serial
 number of the unit which can be found on the unit's
 enclosure). On the BYOD, select this Wi-Fi network.
- Open a Web browser on the BYOD and navigate to http://sn00000.ampsafe (where "00000" is the serial number of the unit which can be found on the unit's enclosure).
- If this does not connect, enter the following numeric IP address in the Web browser: http://192.168.43.1

Once a wired or wireless connection is established and the home page appears in the Web browser, you can get further guidance from the home page by clicking Settings >> Help

If you need further assistance, contact Acertara via Web at https://acertaralabs.com/contact-us/ or via phone at (303)834-8413